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
# **ETV6-RUNX1 RQ (TEL-AML1) Kit Manual**

**For Real-Time PCR Quantitative Detection of  
ETV6-RUNX1 (TEL-AML1) Transcripts**

For use with Rotor-Gene or StepOne  
Research use only

NG-WI-ASL-29-100  
Version 1.1  
Summer 2019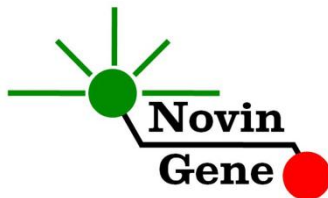

## Table of Contents:

1. Introduction .....	2
2. Kit Contents .....	3
3. Storage and Stability .....	3
4. General Precautions .....	4
5. Additionally Required Materials.....	4
6. Specimen, storage and transport .....	5
7. RNA isolation .....	5
8. cDNA synthesis .....	6
9. PCR Protocol .....	6
10. Programming of RotorGene .....	7
11. Programming of StepOne .....	7
12. Programming of other machines .....	8
13. Data Analysis: RotorGene.....	8
14. Data Analysis: StepOne.....	11
15. ETV6-RUNX1% calculation .....	12
16. Analytical Sensitivity .....	13

**ETV6-RUNX1 (TEL-AML1) RQ** kit is intended for the detection of ETV6-RUNX1 transcripts in peripheral blood and ETV6-RUNX1 percentage calculation in patients undergoing therapy. This kit is designed for use with RotorGene or StepOne machines. This kit is for research use only!

**Important Note:** *This kit doesn't provide reagents for RNA extraction or cDNA synthesis!*

## 1- Introduction

ETV6-RUNX1 (TEL-AML1) is an abnormality resulted from 21;12 translocation. RUNX1 gene, encodes a protein involved in transcriptional control of hematopoiesis. However, as a result of this translocation, it is repressed by ETV6-RUNX1 fusion protein. This alteration occurs in approximately 25% of childhood ALL diagnosed between the ages of 2-10 years.

ETV6-RUNX1 (TEL-AML1) RQ provides a ready-to-use system for detection and quantitation of ETV6-RUNX1 transcripts as well as calculation of ETV6-RUNX1 percentage.

This kit is based on Real-Time PCR technology. In this method application of fluorescent probes allows detection of amplified product. Analysis of fluorescent kinetics also leads to detection of the target sequence in the reaction without requiring post-amplification analysis, reducing the possibility of contamination with the PCR product. This kit is designed to be used with RotorGene or StepOne machines.

## 2- Kit Contents

The kit contains a manual or a quick guide and following reagents:

Label	Content	Quantity
<b>ETV6-RNX1 Mix*</b>	Master mix for ETV6-RUNX1	480 $\mu$ l
<b>ABL Mix*</b>	Master mix for ABL	480 $\mu$ l
<b>ER1</b>	ETV6-RUNX1 Stanndard 1: 100,000 copy/ $\mu$ l	150 $\mu$ l
<b>ER2</b>	ETV6-RUNX1 Stanndard 2: 10,000 copy/ $\mu$ l	150 $\mu$ l
<b>ER3</b>	ETV6-RUNX1 Stanndard 3: 1,000 copy/ $\mu$ l	150 $\mu$ l
<b>ER4</b>	ETV6-RUNX1 Stanndard 4: 100 copy/ $\mu$ l	150 $\mu$ l
<b>ER5</b>	ETV6-RUNX1 Stanndard 5: 10 copy/ $\mu$ l	150 $\mu$ l
<b>A1</b>	ABL Standard 1: 100,000 copy/ $\mu$ l	150 $\mu$ l
<b>A2</b>	ABL Standard 2: 10,000 copy/ $\mu$ l	150 $\mu$ l
<b>A3</b>	ABL Standard 3: 1,000 copy/ $\mu$ l	150 $\mu$ l
<b>A4</b>	ABL Standard 4: 100 copy/ $\mu$ l	150 $\mu$ l
<b>Water</b>	PCR Grade Water	200 $\mu$ l

\* 1, 2 and 4 tubes for 24, 48 and 96 reaction kits.

## 3- Storage and Stability

The kit components should be shipped and stored at  $-20^{\circ}\text{C}$  and are stable until the expiry date mentioned. Avoid repeated freeze-thaws especially for PCR Mix more than few times to prevent reduced sensitivity.

## 4- General Precautions

In order to prevent false results, always pay attention to the following points:

- Treat all samples as potentially infectious.

- Take utmost care to avoid RNase contamination during RNA extraction and cDNA synthesis.
- Within the pre-PCR work area, assign three separate spaces for: a) Sample storage and extraction; b) Reagent preparation where the Master Mix is aliquoted into tubes; and c) Reaction preparation area for addition of templates to the tubes.
- Always wipe the working surfaces with 70% Ethanol before and after work.
- Thaw kit components on ice completely, mix by flickering followed by a quick spin and store on crushed ice after.
- Do not place 0.2 ml PCR tubes on crushed ice. Use cold blocks instead.

## 5- Additionally Required Materials

To use this kit, you need the following items:

- Real-Time PCR machine and accessory computer
- Refrigerated microtube centrifuge
- Vortex Mixer
- Dry Block Heater
- Adjustable pipettors and nuclease free filtered tips
- RNA extraction kit
- cDNA synthesis kit
- Nuclease free 1.7ml microtubes and PCR microtubes
- Disposable powder-free gloves
- Cold block

## 6- Specimen, Storage and Transport

Peripheral blood should be collected in sterile condition in proper and sterile tubes. We recommend EDTA or Citrate as anticoagulant. Whole blood should be shipped and stored at +4°C (stable for 72 hrs).

RNA can directly be extracted from whole blood. Alternately to increase sensitivity, buffy coat can be used. For optimum results, a sample should include 5 to 10 million WBC per 150µl.

Whole blood or buffy coat can be stored at +4°C for three days. Otherwise should be aliquoted and stored at -70°C which is stable for few months.

## 7- RNA Isolation

RNA isolation can be performed using different kits from various manufacturers. We recommend using:

- TriPure isolation Reagent (Cat. no. 1667157, Roche Applied Science, and Mannheim, Germany).
- TRIzol isolation reagent (Cat. no. 15596026, Invitrogene/Thermo Fisher, USA)
- Isol-RNA isolation reagent (Cat. no. 2302700, 5 prime/Thermo Fisher, Germany)
- Accuzol isolation reagent (Cat. no. K-3090, Bioneer, Korea)

## 8- cDNA Synthesis

1µg of total RNA is required and should be reverse transcribed to cDNA using random hexamers or specific primers. Different kits are available in the market for this purpose.

These specific primers are available upon request. If the CT observed for ABL using Random Hexamers is above 27, it is recommended to use specific primers for decreasing CT for ABL and ETV6-RUNX1 gene and increasing sensitivity of the test.

Dissolve RNA pellet in 20ul of the Primers solution and then add other reagents required for cDNA synthesis.

Dilute prepared cDNA 2.5x with nuclease free water. For example to 20ul of cDNA add 30ul of nuclease free water.

## 9- PCR Protocol

Thaw the reagents on ice completely, followed by brief mixing and a quick spin.

Each sample should be examined for both in ETV6-RUNX1 fusion gene and for ABL control gene. So two set of reactions are required. In ETV6-RUNX1 set, consider 1 tube for each sample as well as 6 tubes for the 5 standards (ER1 to ER5) and Negative control or NTC. In ABL set, consider 1 tube for each sample and 5 tubes for the 4 standards (A1 to A4) and Negative control or NTC. Place required number of tubes on cold block.

**Pipette 20µl of ETV6-RNX1 Mix to the first series of tubes and 20µl of ABL Mix to each tube of the second group. Continue by adding 5µl of cDNA or standard or control to each tube.**

Cap the tubes and visually inspect to make sure all are capped securely. Place tubes in the machine.

*Note: Working with StepOne instrument, spin tubes briefly before loading on the block.*

*Note: If using RotorGene attach the locking ring.*

## 10-Programming RotorGene

*Before you start the machine, make sure you have attached the locking ring on the rotor!*

Open the CD provided in the kit and double click on “ETV6-RUNX1 0.2” or on “ETV6-RUNX1 Strip” depending to tubes used. Click on Start button (Green button on the top menu). On the pop up window click start again and save the run file.

Edit sample names on both ETV6-RUNX1 and ABL pages. Remember that tubes containing ETV6-RNX1 Mix should only be named in ETV6-RUNX1 page and tubes containing ABL Mix should only be named in ABL page.

Make sure in the “Type” column, all the standards have been defined as “standard” and the relative concentrations have been entered. Patient samples should be defined as "unknown" and no template control as "NTC", respectively.

## 11-Programming of StepOne

Open the StepOne software (V 2.\*). On the Set Up menu click on Template and select the file on CD provided with the kit. Click on Plate Setup. One negative control, 5 standards for ETV6-RUNX1, 4 standards for ABL and few samples are defined. You may change plate set up using right click options (copy, past, clear). You may also add or remove samples on “Define Targets and Samples” menu. When finished click on Start Run and save the experiment. Instrument will start shortly.

## 12-Programming Other Machines

If you apply this kit to other Real-Time PCR machines, program it according to the following table:

Step	Temperature and time	Cycles
1	<b>95°C x 10 min</b>	1
2	<b>95°C x 15 sec</b>	45
	<b>60°C x 60 sec</b>	

Fluorescence should be collected at 60C for FAM and VIC dyes. Both of ETV6-RUNX1 Mix and ABL Mix contain ROX with final concentration of 300nM in reaction.

## 13-Data Analysis: RotorGene

Before analyzing results, make sure in the sample menu all the standards have been defined as “standard” and the relative concentrations have been entered. Patient samples should be defined as "unknown" and Negative control or no template control as "Negative Control" or "NTC", respectively.


Analyze the data according to manufacturer recommendations. Perform quantitative analysis for both **ETV6-RUNX1 (Green channel)** and **ABL (Yellow channel)**. Briefly, click on analysis menu and then, under Quantitation tab double click on “Cycling A. Green”.

Close the pop up window and manually set threshold at 0.1.


Repeat above for Yellow channel and set the threshold at 0.1.

Figures 1 and 2 represent typical graphs for RotorGene machine.

Consider following points when analyzing:

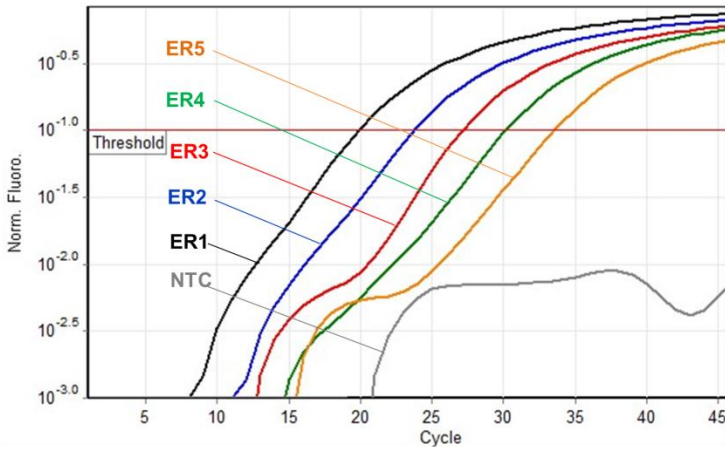**Fig 1.** Typical ETV6-RUNX1 graph in Green channel for Rotor-Gene

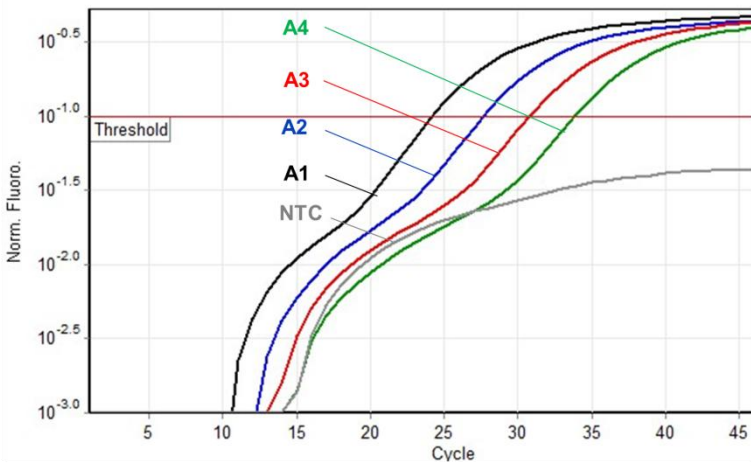**Fig 2.** Typical ABL graph in Yellow channel for Rotor-Gene

**Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.**

**In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.**


- A sample is **Positive** if it is positive in both Green/ETV6-RUNX1 and Yellow/ABL channels with sigmoid graphs and CT of 20-40 for Green and 20-30 for Yellow.
- A sample is **Negative** if is negative in Green/ETV6-RUNX1 channel while it is positive in Yellow/ABL channel with a sigmoid graph and CT of 20-27.
- Results are **inconclusive** and the test should be repeated if a sample is negative in both of Green/ETV6-RUNX1 and Yellow/ABL channels. Improper extraction or test set up could cause that.
- Results are **inconclusive** and the test should be repeated if a sample is negative in Green/ETV6-RUNX1 channel while it is positive in Yellow/ABL channel with sigmoid graph and CT of above 27. Improper extraction or low RNA input could cause that.

*Note: All patient samples should be positive in Yellow/ABL channel with a sigmoid graph and CT of 27 or less. ABL CT greater than 27 usually happens if not enough cells have been extracted or less than 100ug RNA has been used. CTs higher than 27 reduce the sensitivity of the test and may result in **false negative** reports.*


### 14-Data Analysis: StepOne

Analyze the data according to manufacturer recommendations. Briefly, click on “Analyze” and set the threshold for both **ETV6-RUNX1/FAM** and **ABL/VIC** on 0.1.

Figures 3 and 4 represent typical graphs for StepOne machine.

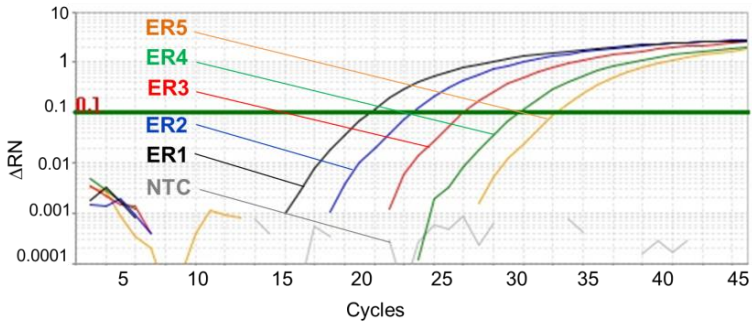**Fig3.** Typical ETV6-RUNX1 graph in FAM channel for StepOne

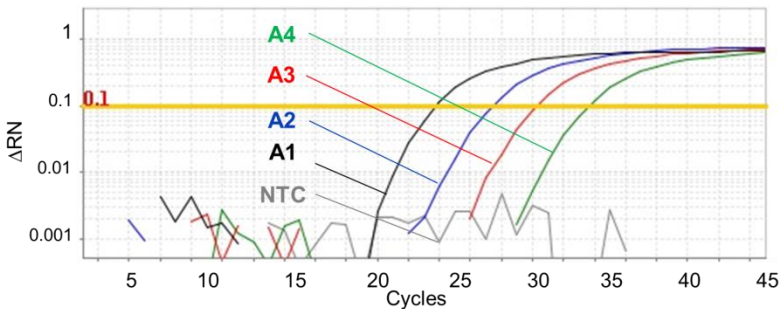**Fig 4.** Typical ABL graph in VIC channel for StepOne

**Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.**

**In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.**

Consider following points when analyzing:

- A sample is **Positive** if it is positive in both FAM/ETV6-RUNX1 and VIC/ABL channels with sigmoid graphs and CT of 20-40 for Green and 20-30 for Yellow.
- A sample is **Negative** if it is negative in FAM/ETV6-RUNX1 channel while it is positive in VIC/ABL channel with a sigmoid graph and CT of 20-27.
- Results are **inconclusive** and the test should be repeated if a sample is negative in both of FAM/ETV6-RUNX1 and VIC/ABL channels. Improper extraction or test set up could cause that.
- Results are **inconclusive** and the test should be repeated if a sample is negative in FAM/ETV6-RUNX1 channel while it is positive in VIC/ABL channel with sigmoid graph and CT of above 27. Improper extraction or low RNA input could cause that.

*Note: All patient samples should be positive in VIC/ABL channel with a sigmoid graph and CT of 27 or less. ABL CT greater than 27 usually happens if not enough cells have been extracted or less than 100ug RNA has been used. CTs higher than 27 reduce the sensitivity of the test and may result in **false negative** reports.*

## **15-ETV6-RUNX1% calculation**

To assess the response to therapy, ETV6-RUNX1% value for each patient can be calculated. This kit uses NCN method for this purpose (Beillard E. 2003, *Leukemia* 17:2474). In this method

ETV6-RUNX1% value is the ETV6-RUNX1 transcripts (titer) normalized by the ABL transcripts (titer) and then multiplied by 100%.

Usually titer of ABL gene transcripts is higher than for ETV6-RUNX1 fusion gene. Therefore results of above calculation would be a number less than 100%. This value may fall even below 0.001% in optimum cases. However at diagnosis and in case of relapse, ETV6-RUNX1 transcripts may surpass ABL gene transcripts. As a result, ratio of ETV6-RUNX1/ABL would be above 100%. Such results have been reported previously too. For example please note table 12 of the paper published by J. Gabert *et al* (2003, *Leukemia* 17:2318) for B-ALL patients. He reported a range of 20% to 1520% with average of 471% for peripheral blood and range of 46% to 3270% with average of 445% for bone marrow samples. Therefore a ratio above 100% is not unexpected and simply means higher transcripts of the fusion gene compared to the control gene.

## **16-Analytical Sensitivity**

Analytical sensitivity of the kit has been determined by examining dilution series of cloned cDNA and was estimated about 2 copy/ul.