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
# **HCV RQ Kit Manual**

**For Real-Time PCR Detection and  
Quantitation of Hepatitis C Virus  
(HCV) RNA**

For use with Rotor-Gene or StepOne  
Research use only

NG-WI-ASL-02-400  
Version 4.0  
April 2018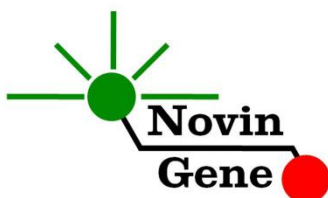

## Table of Contents:

1. Introduction.....	2
2. Kit Contents.....	3
3. Storage and Stability .....	3
4. General Precautions .....	3
5. Additionally Required Materials .....	4
6. Specimen, storage and transport.....	4
7. Interfering substances .....	5
8. RNA isolation.....	5
9. Internal control (IC).....	5
10. Quantitation.....	5
11. RT-PCR Protocol.....	6
12. Programming of the Rotor-Gene .....	6
13. Programming of StepOne .....	7
14. Programming other machines.....	7
15. Data Analysis: Rotor-Gene .....	9
16. Data Analysis: StepOne.....	11
17. Linear Range.....	12
18. Sensitivity .....	12

**HCV RQ** kit is intended for the quantitative detection of HCV-RNA extracted from plasma. It is designed for use with Rotor-Gene or StepOne machines. This kit is for research use only.

## 1. Introduction

Hepatitis C virus (HCV) is an RNA virus and an important human pathogen. It is estimated that about 180 million individuals are infected worldwide. It can cause acute and chronic hepatitis followed by liver cirrhosis and hepatocellular carcinoma.

Currently infection can be diagnosed by detection of HCV RNA through RT-PCR in patient sample, weeks to months before antibody response. In addition, monitoring viral load by the same method has become a clinical gold standard for evaluation of therapy success as well as disease progress.

HCV RQ kit provides a ready-to-use One-Step Real-Time RT-PCR system for detection and quantitation of HCV RNA with Rotor-Gene or StepOne machines. Currently the Real-Time RT-PCR provides the highest sensitivity and widest dynamic range among other methods. In this method application of fluorescent probes allows detection of amplified product. Analysis of fluorescent kinetics also leads to quantification of the target sequence in the reaction without requiring post-amplification analysis therefore, reducing the possibility of contamination with the PCR product. This kit also incorporates an *Internal Control (IC)* to identify possible RNA extraction failure or PCR inhibition.

## 2. Kit Contents

The kit contains a manual, a CD with Rotor-Gene and StepOne templates and following reagents:

Label	Content	Quantity
HCV Mix	RT-PCR Master mix	360 $\mu$ l*
HCV S1	Standard 1: 50,000 IU/ul	150 $\mu$ l
HCV S2	Standard 2: 5,000 IU/ul	150 $\mu$ l
HCV S3	Standard 3: 500 IU/ul	150 $\mu$ l
HCV S4	Standard 4: 50 IU/ul	150 $\mu$ l
HCV IC	Internal Control	250 $\mu$ l*
Water	PCR Grade Water	200 $\mu$ l

\* 1, 2 and 4 tubes for 24, 48 and 96 reaction kits respectively.

## 3. Storage and Stability

The kit components should be shipped and stored at  $-20^{\circ}\text{C}$  and are stable until the expiry date mentioned. Avoid repeated freeze-thaws especially for HCV Mix more than few times to prevent reduced sensitivity.

## 4. General Precautions

In order to prevent false results, always pay attention to the following points:

- Treat all samples as potentially infectious.
- Within the pre-PCR work area assign three separate spaces for: a) Sample storage and extraction, b) Reagent preparation where the master-mix is aliquoted into tubes and c) Reaction preparation area for addition of extracted RNA to the tubes.
- Always wipe the working surfaces with 70% Ethanol before and after work.

- Thaw on ice kit components completely, mix by flickering followed by a quick spin and store on crushed ice after.
- Keep RT-PCR Mix tube at  $-20^{\circ}\text{C}$  at all times. Take it out just before use and return it to freezer immediately after.
- Do not place 0.2ml PCR tubes on crushed ice. Use cold block instead.

## 5. Additionally Required Materials

To use this kit, you need the following items:

- Real-Time PCR machine and accessory computer
- Table top microtube centrifuge
- Vortex Mixer
- Dry Block Heater
- Adjustable pipetters and nuclease free filtered tips
- RNA extraction kit
- Nuclease free 1.7ml microtubes and PCR microtubes
- Disposable powder-free gloves
- Cold block

**This kit does not require cDNA synthesis reagents!**

## 6. Specimen, Storage and Transport

We recommend EDTA or citrate plasma for HCV detection. Peripheral blood should be collected in sterile condition in proper and sterile tubes. Whole blood or plasma should be shipped at  $+4^{\circ}\text{C}$ . Upon receipt plasma should be separated from whole blood and can be stored at  $+4^{\circ}\text{C}$  for few days or aliquoted and stored at  $-20^{\circ}\text{C}$  for up to few weeks.

## 7. Interfering Substances

Heparin (more than 10 IU/ml) affects the PCR. Blood collected in heparin containing tubes should not be used. Samples of

heparinized patients must not be used as well. Elevated levels of bilirubin ( $\leq 4.5$  mg/dl) and lipids ( $\leq 1000$  mg/dl) and hemolytic samples do not influence the extraction and PCR.

## 8. Internal Control (IC)

In order to evaluate the possibility of RNA extraction failure and PCR inhibition and prevent false negative results, HCV RQ kit contains internal control (IC). IC can be used during extraction process or simply added to HCV Mix.

To monitor both RNA extraction and PCR reaction, internal control should be added to the mixture of lysis buffer and patient sample. Required volume of internal control is 10% of elution buffer. For instance, if extracted RNA is eluted with 100ul, then 10ul of internal control should be added to the mixture of lysis buffer and patient sample. **Please note that internal control should not be added directly to the patient sample (i.e. before addition of lysis buffer) as it loses its efficiency.**

If IC is added to HCV Mix, only PCR inhibition can be monitored. For this purpose, 1ul of IC reagent should be added to each reaction. For example, for 10 PCR reactions, 15ul of internal control should be added to 150ul of HCV Mix before it is added to the tubes.

In a successful RNA extraction and PCR test, IC should generate a CT of 31-37 in Yellow/VIC Channel.

## 9. RNA Isolation

RNA isolation can be performed using different kits from various manufacturers. We recommend the following:

- High Pure Viral Nucleic Acid Kit (Cat. no. 11858874001, Roche Applied Science, Mannheim, Germany)

- QIAamp Viral RNA Mini Kit (Cat. no. 52904, Qiagen GmbH, Hilden, Germany)
- QIAamp UltraSens ® Virus Kit (Cat. no. 53704, Qiagen GmbH, Hilden, Germany)
- QIAamp MiniElute Virus Spin Kit (Cat. no. 57704, Qiagen GmbH, Hilden, Germany).

To monitor RNA extraction process, internal control should be applied to the extraction process. For more details, please, refer to section 8 of this handbook

## 10. RT-PCR Protocol

Thaw the reagents on ice completely followed by a brief mixing and a quick spin. Place required number of tubes cold block. Consider one tube for each sample plus one for each standard and one for the negative control.

**If IC is introduced during extraction process, pipette 15µl of HCV Mix directly to each PCR tube.**

**If IC is added to the HCV Mix, add 15ul of the prepared mix (as described in section 8) to each PCR tube.**

**Then add 10ul of extracted RNA, **standard** or water to each tube.**

Cap the tubes and visually inspect to make sure all are capped securely. Place tubes in the machine.

*Note: Working with StepOne instrument, spin tubes briefly before loading on the block.*

*Note: If using Rotor-Gene attach the locking ring too.*

## 11. Programming of the Rotor-Gene

*Before you start the machine, make sure you have attached the locking ring on the rotor!*

Open the CD provided in the kit and double click on "HCV 0.2" or on "HCV 0.1" depending on the tubes used. Program starts. Click on Start button (Green button on the top menu). On the pop up window click start again and save program on desired location.

## 12. Programming of StepOne

Open the StepOne software (V 2.\*). On the Set Up menu click on Template and select the file on CD provided with the kit. Click on Plate Setup. One negative control, 4 standards and 10 samples are defined. You may change plate set up using right click options (copy, paste, clear). You may also add/remove samples or change sample name on "Define Targets and Samples" menu. When finished, click on "Start Run" and save the experiment on desired location. Instrument will start shortly.

## 13. Programming Other Machines

If you apply this kit to other Real-Time PCR machines, program it according to the following table:

Step	Temperature and time	Cycles
1	<b>50°C x 30 min</b>	1
2	<b>95°C x 15 min</b>	1
3	<b>95°C x 15 sec</b>	45
	<b>60°C x 60 sec</b>	

Fluorescence should be collected at 60°C for FAM and VIC dyes. The HCV Mix contains ROX. Final concentration of ROX in the reaction is 300nM.

## 14. Data Analysis: Rotor-Gene


Before analyzing results, make sure in the sample menu all the standards have been defined as "standard" and the relative concentrations have been entered. Patient samples should be defined as "unknown". Name Negative control or no template control as "Negative Control" or "NTC", respectively.

Analyze data according to manufacturer recommendations. Perform quantitative analysis for **HCV (Green channel)** and qualitative analysis for **Internal Control (Yellow channel)**. Briefly, click on analysis menu and then under Quantitation tab double click on cycling A. Green.


In the pop up for Automatic Threshold increase the minimum or lower bound until it surpasses the negative control or the NTC fluorescence, and then click on OK. Alternatively, you may simply set threshold at 0.1.

Repeat the above for Cycling A. Yellow, cancel the Automatic Threshold and manually put threshold on 0.1.

Figures 1 and 2 represent typical graphs for Rotor-Gene machine.

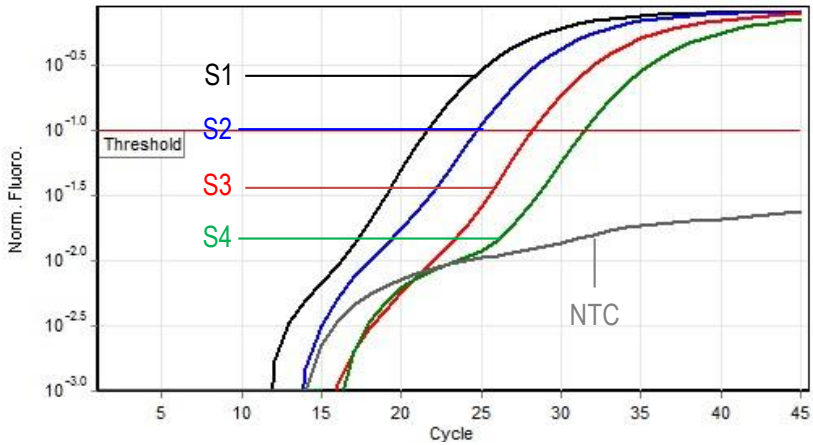**Figure 1:** Typical HCV graph in green channel for Rotor-Gene instrument.

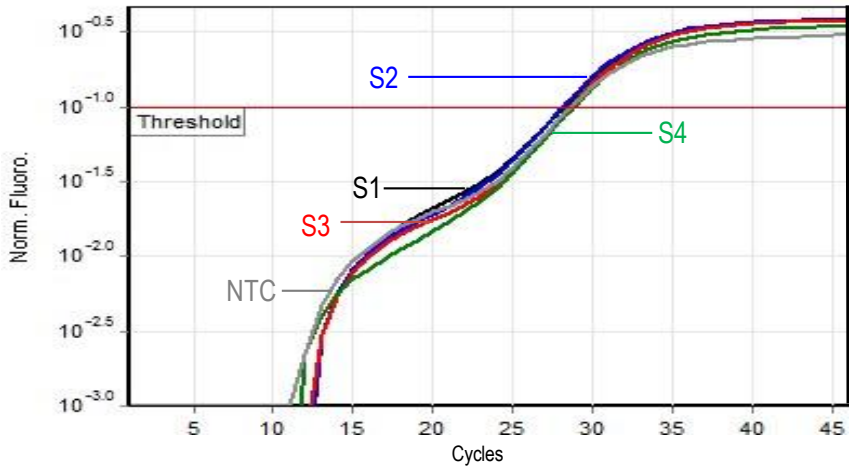**Figure 2:** Typical IC graph in yellow channel for Rotor-Gene instrument.

**Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.**

**In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.**


Consider following points when analyzing:

- A sample is **Positive** if it is positive in Green channel with a sigmoid graph and a CT of less than 40. The viral load or quantitation results in the Cycling A. Green are valid.
- A sample is **Negative** if it is negative in Green channel while it is positive in Yellow channel with a sigmoid graph and CT of 28-34.
- Results are **Inconclusive** and the test should be repeated if a sample is negative in both of Green and Yellow channels.


## 15. Data Analysis: StepOne

Analyze data according to the manufacturer's recommendations. Briefly, click on Analyze and set the threshold for **HCV/FAM** at 0.1 and at 0.05 for **IC/VIC**.

Figures 3 and 4 represent typical graphs for StepOne machine.

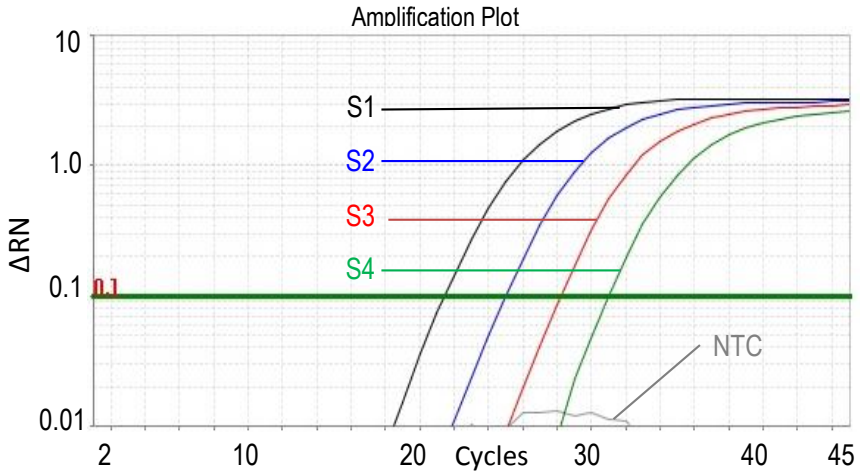**Figure 3:** Typical HCV graph in FAM channel for StepOne instrument.

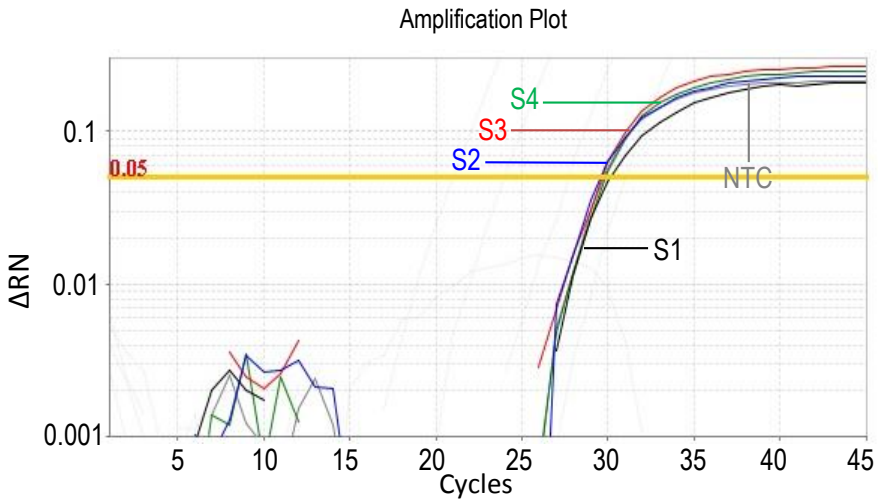**Figure 4:** Typical IC graph in VIC channel for StepOne instrument.

**Note that a sample is considered Positive only if it has a sigmoid graph and log phase, and only then CT is reliable and can be used.**

**In the absence of sigmoid graph and log phase, sample is considered Negative and CT if present is not reliable.**

Consider following points when analyzing:

- A sample is **Positive** if it is positive in FAM/HCV channel with a sigmoid graph and a CT of less than 40. The viral load or quantitation results are valid.
- A sample is **Negative** if it is negative in FAM/HCV channel while it is positive in VIC/IC channel with a sigmoid graph and CT of 28-34.
- Results are **Inconclusive** and the test should be repeated if a sample is negative in both of FAM/HCV and VIC/IC channels.

## 16. Quantitation

The kit provides 4 quantitation standards with defined titers to generate a standard curve for quantification of samples viral load. Working with Rotor-Gene machine, the standard curve from a previous run can also be imported to the recent run. To do so, at least one standard must be used in the current run. Apparently using all four standards in each run will lead to more accurate results.

Quantitation standards are defined as IU/ $\mu$ l. To convert the result to IU/ml following equation should be used:

$$\text{Result (IU/ml)} = \frac{\text{Result (IU/}\mu\text{l)} \times \text{elution volume (}\mu\text{l)}}{\text{sample volume (ml)}}$$

“Sample volume” is the plasma volume used for RNA isolation and “Elution volume” is the volume of buffer or water used to elute or dissolve isolated RNA.

### **17. Linear Range**

The linear range of the kit was assessed with dilution series of the cloned target and showed to be linear in the range of 2,500,000 IU/ $\mu$ l to 5 IU/ $\mu$ l.

### **18. Sensitivity**

The analytical detection limit of the kit was assessed with dilution series of the cloned target and showed a limit of detection equal to 1.5 IU/ $\mu$ l.